


# Plan History

The *Plan History* allows the user to see all changes and modifications made to a plan. Each modification to a plan will result in a new *Revision* of the plan (note: in a complex plan there may be hundreds of revisions). If the user is not satisfied with the results of a modification they may go back to any of the previous *Revisions* and make that instance the active instance (note: this will cause another *Revision* to be created for the reversion). Below is a walk through and more detailed explanation of this feature.

1. Open the *General* block from the Plan Overview
2. Expand the *History* tree node
  1. The number of plan revisions will be listed (In the below example there are 36 revisions)
3. Select the *More Details* option in the bottom right hand corner

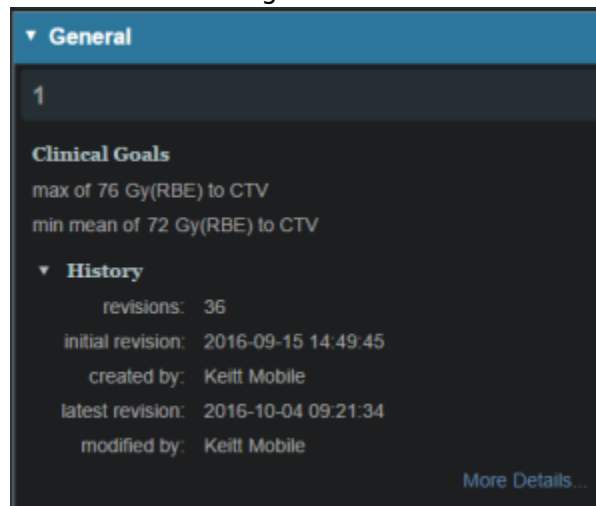

1. The summary of the latest revisions will expand to the right

General	Revision 36: view make active
1	created: 2016-10-04 09:21:34
	by: Keitt Mobile
	summary: show
Clinical Goals	Revision 35: view make active
max of 76 Gy(50%) to CTV	created: 2016-10-03 10:54:29
max mean of 72 Gy(45%) to CTV	by: Keitt Mobile
	summary: show
History	Revision 34: view make active
revisions: 36	created: 2016-09-19 09:47:29
initial revision: 2016-09-15 14:09:40	by: Keitt Mobile
created by: Keitt Mobile	summary: show
latest revision: 2016-10-04 09:21:34	Revision 33: view make active
modified by: Keitt Mobile	created: 2016-09-19 09:32:52
	by: Keitt Mobile
	summary: show
	Revision 32: view make active
	created: 2016-09-19 09:28:54
	by: Keitt Mobile
	summary: show
Patient Geometry	Revision 31: view make active
15 segmented structures	created: 2016-09-19 09:47:39
1 planning structure	by: Keitt Mobile
	summary: show
RSP Image	Revision 30: view make active
0 structure(s) have density overrides	created: 2016-09-16 10:57:22
	by: Keitt Mobile
	summary: show
Calculation Grid	Revision 29: view make active
6 user Dose Spacing	created: 2016-09-16 08:44:43
2 Grid Regions	by: Keitt Mobile
	summary: show
Beams	Revision 28: view make active
0 plan beams	created: 2016-09-16 08:31:08
	by: Keitt Mobile
	summary: show
Optimization	Revision 27: view make active
4 fraction groups	created: 2016-09-16 08:30:35
6 optimization constraints	by: Keitt Mobile
14 optimization objectives	summary: show
	Revision 26: view make active
	created: 2016-09-16 08:30:08
	by: Keitt Mobile
	summary: show
	Revision 25: view make active
	created: 2016-09-16 08:31:08
	by: Keitt Mobile
	summary: show
	Revision 24: view make active
	created: 2016-09-15 17:44:36
	by: Keitt Mobile
	summary: show
	Revision 23: view make active
	created: 2016-09-15 17:44:12
	by: Keitt Mobile
	summary: show
	Revision 22: view make active
	created: 2016-09-15 15:05:09
	by: Keitt Mobile
	summary: show

4. The user can now review the various revision summaries

1. Clicking the blue *show* link will expand that particular version of the plan and explain what was changed
2. “op”: is the operation performed in this version that differs from the previous version (i.e. delete: an item was deleted)
3. “path”: is the item in the treatment plan data that had the operation performed
4. “val”: is the value in the path that had the operation performed

5. If the user would like to return to this version they may do so by clicking the blue *make active* link

6. The user can close the expanded version by clicking the blue *hide* link

Revision 36: view make active
created: 2016-10-04 09:21:34
by: Keitt Mobile
summary: hide
{
op: update,
path: [ captured scope, directive ],
val: {
some: 57f2736f00c0b9f2c378152e06cfd09
}
{
op: update,
path: [ captured scope, site ],
val: {
some: 57dbebec00c0102cd41ea86de124b527
}
}

From:

<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:

[http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:plan\\_history&rev=1477321173](http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:plan_history&rev=1477321173)

Last update: **2021/07/29 18:25**

