



# Plan History

The *Plan History* allows the user to see all changes and modifications made to a plan. Each modification to a plan will result in a new *Revision* of the plan (note: in a complex plan there may be hundreds of revisions). If the user is not satisfied with the results of a modification they may go back to any of the previous *Revisions* and make that instance the active instance (note: this will cause another *Revision* to be created for the reversion). Below is a walk through and more detailed explanation of this feature.

1. Open the *General* block from the Plan Overview

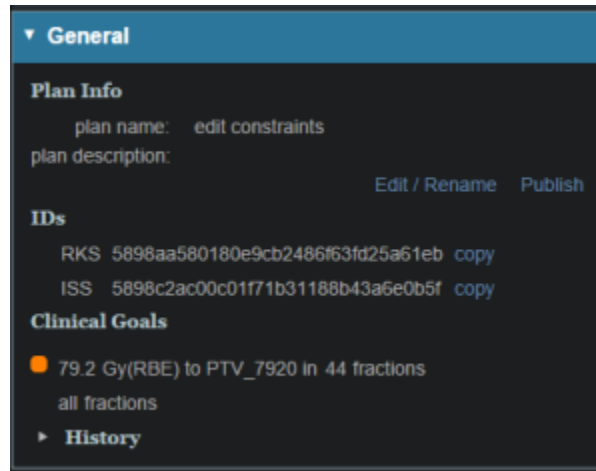

2. The summary of the latest revisions will expand to the right

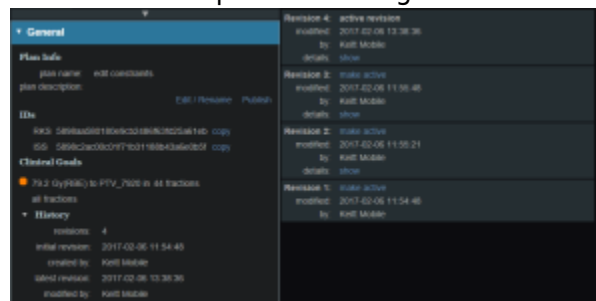

3. Expand the *History* (this is not mandatory)
  1. The number of plan revisions will be listed (In the above example there are 4 revisions)
4. The user can now review the various revision summaries
  1. Clicking the blue *show* link will expand that particular version of the plan and explain what was changed
  2. “op”: is the operation performed in this version that differs from the previous version (i.e. delete: an item was deleted)
  3. “path”: is the item in the treatment plan data that had the operation performed
  4. “val”: is the value in the path that had the operation performed
5. If the user would like to return to this version they may do so by clicking the blue *make active* link
6. The user can close the expanded version by clicking the blue *hide* link

```
Revision 36: view make active
created: 2016-10-04 09:21:34
by: Keitt Mobile
summary: hide
{
  op: update,
  path: [ captured scope, directive ],
  val: {
    some: 57f2736f00c0b9f2c378152e06cfd09
  }
}
{
  op: update,
  path: [ captured scope, site ],
  val: {
    some: 57dbebec00c0102cd41ea85de124b527
  }
}
```

From:  
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:  
[http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:plan\\_history&rev=1487264003](http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:plan_history&rev=1487264003)

Last update: **2021/07/29 18:25**

