


Plan Templates

Templates can be saved at either the site or user level. **All** *Italic Text* users will be able to access site level templates. Only the user who created them will be able to use user level templates. Templates must be validated prior to creating a new treatment plan to ensure compatibility with the new patient model

1. Click on the blue *Add Plan* link under the *Patient Model* to create a new plan
2. Choose the plan that should be used from the Base Plan drop down menu and click blue OK

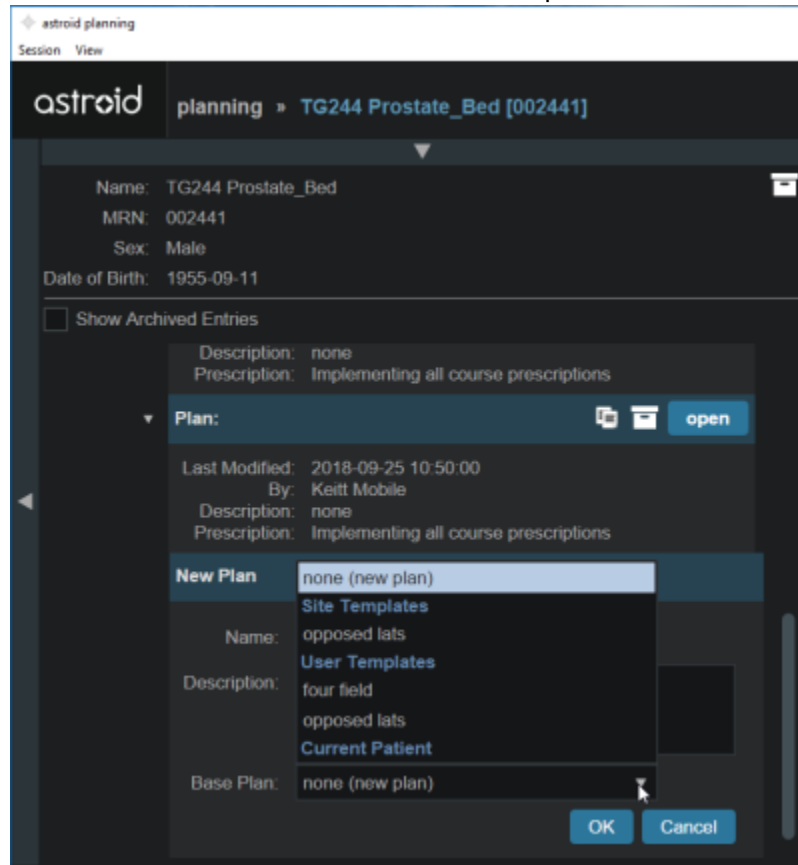

The screenshot shows the 'astroid planning' interface for a patient named 'TG244 Prostate_Bed [002441]'. The patient's details include MRN: 002441, Sex: Male, and Date of Birth: 1955-09-11. The 'Plan' dropdown menu is open, showing options: 'none (new plan)', 'Site Templates', 'User Templates', 'four field', 'opposed lats', and 'Current Patient'. The 'Base Plan' dropdown is also visible, showing 'none (new plan)'. There are 'OK' and 'Cancel' buttons at the bottom right.

3. Make any changes necessary within plan and proceed with planning

From:
<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:
http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:plan_templates&rev=1537977187

Last update: **2021/07/29 18:25**

