2025/11/03 10:49 1/2 Plan Templates

## **Plan Templates**

Plan Templates are a powerful feature of Astroid that can greatly reduce the time spent in creating high quality treatment plans. In a simple sense, a Plan Template is simply a starting point that can be used when creating new treatment plans for this or other patients. When a Plan Template is applied during new plan creation (see here for creating a new plan), all user specified plan parameters from the template are copied into the new plan. In essence you are applying the plan to a new set of images and structures, therefore it is important that structure names and other data are consistent in your Patient Models when using Plan Templates.

## Saving Plan Templates

*Plan Templates* can be defined globally at the site level for all users at your institution and each user is also free to create their own private list of user level *Plan Templates*. A *Plan Template* can be easily created from any existing plan by ...


When saving an existing plan as a template, there are a few rules/tips that should be kept in mind which help make your templates easier to use in the future ...

- Tip 1
- Tip 2
- Templates must be validated prior to the creating a new treatment plan to ensure compatibility with the new patient model

## **Using Plan Templates**

Plan Templates are applied when creating a new plan:

- 1. Click on the blue Add Plan link under the Patient Model to create a new plan
- 2. From the Base Plan drop down menu choose the *Plan Template* that should be used and then click blue OK

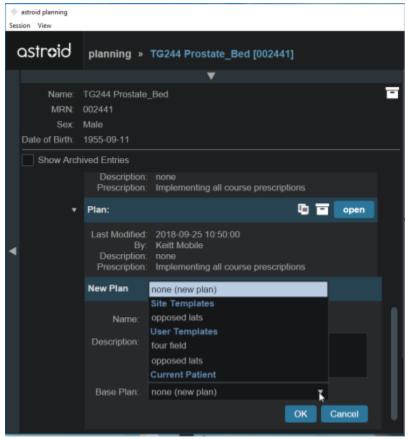

- 3. Open the new plan and check that all plan parameters have been applied as expected
- 4. Make any changes necessary within plan and then proceed with planning from this starting point
- 5. If used well, you should be able to review the plan, make a few minor edits, and be at the point of running feasibility & optimization within just a few minutes

From:

http://apps.dotdecimal.com/ - decimal App Documentation

Permanent link:

http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:plan\_templates&rev=1537978263

Last update: 2021/07/29 18:25

