



RSP Image

The *RSP Image* block allows the user to choose (if not chosen during patient import) or change the HU (Hounsfield Unit) to RSP (Relative Stopping Power) curve to use during the planning process. The user may also choose any density overrides for a particular structure at this point. An example of when to choose an override is the case of hip prosthesis. The user will need to know the RSP to be used for the chosen structure.


1. Open the *RSP Image* block
2. Choose the HU to RSP curve (if not chosen at patient import) from the dropdown list

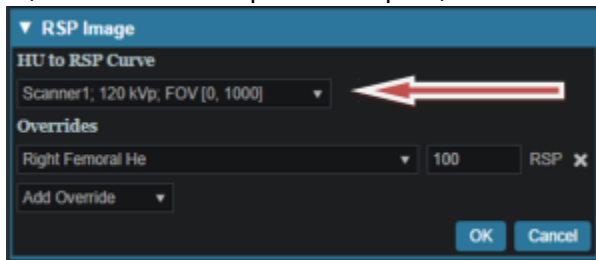

3. If a structure needs a density to be overridden choose that structure from the *Add Override* dropdown list

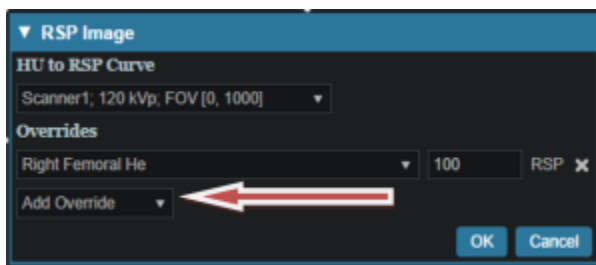

4. Enter the RSP value for the chosen structure

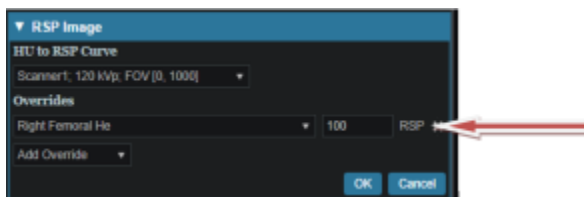

5. Click *OK* when done

From:

<http://apps.dotdecimal.com/> - decimal App Documentation

Permanent link:

http://apps.dotdecimal.com/doku.php?id=planning:userguide:tutorials:rsp_image&rev=1498587177

Last update: 2021/07/29 18:25

