## **Astroid App Keyboard & Mouse Controls**

The following document details the shortcut keys provided in the user interface of the Astroid Planning App. The shortcuts are detailed in their appropriate section of use.

## **Mouse Controls**

Display Views	
Key	Description
Double Left Click	Reset Zoom (fit to scene display)
Right Click + Drag Up	Zoom in
Right Click + Drag Down	Zoom out
Middle Click + Drag	Pan
Left Click + Drag	Rotate (3D only)

## **Keyboard Controls**

	General	
Key	Description	
[	Toggle expansion of left side task menu	
]	Toggle expansion of right side task menu	
Ctrl + =	Increase font size	
Ctrl + +	Increase font size	
Ctrl + -	Decrease font size	
Ctrl + 0	Reset font size to default	
F5	Manual Forced refresh of application and data	
F11	Toggle full screen view	
3D Views		
Key	Description	
Ctrl + 1	Reset the camera to view from the front perspective	
Ctrl + 2	Reset the camera to view from the left perspective	
Ctrl + 3	Reset the camera to view from the right perspective	
Ctrl + 4	Reset the camera to view from the back perspective	
Ctrl + 5	Reset the camera to view from the top perspective	
Ctrl + 6	Reset the camera to view from the bottom perspective	
Ctrl + 7	Reset the camera to view from the isometric perspective	
Ctrl + S	Toggle the structure view type between solid and contour slice	
Aperture Editing		
Key	Description	
Ctrl + Z	Undo the last edit (only tracks edits to undo while in the same instance of the task)	
ESC	Stop drawing manual edits (during line drawing)	

General	
Optimization Navigation	
Key	Description
Left/Right Arrow	Move mco slider in 1/10 increment (applies to last changed slider)

From:

http://apps.dotdecimal.com/ - decimal App Documentation

Permanent link:

http://apps.dotdecimal.com/doku.php?id=planning:userguide:userguide:keyboard\_controls

Last update: 2021/07/29 18:28

