

# Astroid App Keyboard & Mouse Controls

The following document details the shortcut keys provided in the user interface of the Astroid Planning App. The shortcuts are detailed in their appropriate section of use.

## Mouse Controls

Display Views	
Key	Description
Double Left Click	Reset Zoom (fit to scene display)
Right Click + Drag Up	Zoom out
Right Click + Drag Down	Zoom in
Middle Click + Drag	Pan
Left Click + Drag	Rotate (3D only)

## Keyboard Controls

General	
Key	Description
[	Toggle expansion of left side task menu
]	Toggle expansion of right side task menu
Ctrl + =	Increase font size
Ctrl + +	Increase font size
Ctrl + -	Decrease font size
Ctrl + 0	Reset font size to default
F5	Manual Forced refresh of application and data
F11	Toggle full screen view
3D Views	
Key	Description
Ctrl + 1	Reset the camera to view from the front perspective
Ctrl + 2	Reset the camera to view from the left perspective
Ctrl + 3	Reset the camera to view from the right perspective
Ctrl + 4	Reset the camera to view from the back perspective
Ctrl + 5	Reset the camera to view from the top perspective
Ctrl + 6	Reset the camera to view from the bottom perspective
Ctrl + 7	Reset the camera to view from the isometric perspective
Ctrl + S	Set the structure view type between solid and contour slice
Aperture Editing	
Key	Description
Ctrl + Z	Undo the last edit (only tracks edits to undo while in the same instance of the task)
ESC	Stop drawing manual edits (during line drawing)

<b>General</b>	
<b>Optimization Navigation</b>	
<b>Key</b>	<b>Description</b>
Left/Right Arrow	Move mco slider in 1/10 increment (applies to last changed slider)

From:

<http://apps.dotdecimal.com/> - **decimal App Documentation**

Permanent link:

[http://apps.dotdecimal.com/doku.php?id=planning:userguide:userguide:keyboard\\_controls&rev=1484757006](http://apps.dotdecimal.com/doku.php?id=planning:userguide:userguide:keyboard_controls&rev=1484757006)

Last update: **2021/07/29 18:25**

