

# Astroid App Keyboard & Mouse Controls

The following document details the shortcut keys provided in the user interface of the Astroid Planning App. The shortcuts are detailed in their appropriate section of use.

## Mouse Controls

Display Views	
Key	Description
Double Left Click	Reset Zoom (fit to scene display)
Right Click + Drag Up	Zoom out
Right Click + Drag Down	Zoom in
Middle Click + Drag	Pan
Left Click + Drag	Rotate (3D only)

## Keyboard Controls

General	
Key	Description
[	Toggle expansion of left side task menu
]	Toggle expansion of right side task menu
Ctrl + =	Increase font size
Ctrl + +	Increase font size
Ctrl + -	Decrease font size
Ctrl + 0	Reset font size to default
F5	Manual Forced refresh of application and data
F11	Toggle full screen view
3D Views	
Key	Description
Ctrl + 1	Reset the camera to view from the front perspective
Ctrl + 2	Reset the camera to view from the left perspective
Ctrl + 3	Reset the camera to view from the right perspective
Ctrl + 4	Reset the camera to view from the back perspective
Ctrl + 5	Reset the camera to view from the top perspective
Ctrl + 6	Reset the camera to view from the bottom perspective
Ctrl + 7	Reset the camera to view from the isometric perspective
Ctrl + S	Set the structure view type between solid and contour slice
Aperture Editing	
Key	Description
Ctrl + Z	Undo the last edit (only tracks edits to undo while in the same instance of the task)
ESC	Stop drawing manual edits (during line drawing)

General	
Optimization Navigation	
Key	Description
Left/Right Arrow	Move mco slider in 1/10 increment (applies to last changed slider)

From:

<http://apps.dotdecimal.com/> - decimal App Documentation

Permanent link:

[http://apps.dotdecimal.com/doku.php?id=planning:userguide:userguide:keyboard\\_controls&rev=1484757006](http://apps.dotdecimal.com/doku.php?id=planning:userguide:userguide:keyboard_controls&rev=1484757006)

Last update: **2021/07/29 18:25**

