## **Astroid App Keyboard & Mouse Controls**

The following document details the shortcut keys provided in the user interface of the Astroid Planning App. The shortcuts are detailed in their appropriate section of use.

## **Mouse Controls**

Display Views	
Key	Description
Double Left Click	Reset Zoom (fit to scene display)
Right Click + Drag Up	Zoom in
Right Click + Drag Down	Zoom out
Middle Click + Drag	Pan
Left Click + Drag	Rotate (3D only)

## **Keyboard Controls**

	General		
Key	Description		
[	Toggle expansion of left side task menu		
]	Toggle expansion of right side task menu		
Ctrl + =	Increase font size		
Ctrl + +	Increase font size		
Ctrl + -	Decrease font size		
Ctrl + 0	Reset font size to default		
F5	Manual Forced refresh of application and data		
F11	Toggle full screen view		
3D Views			
Key	Description		
Ctrl + 1	Reset the camera to view from the front perspective		
Ctrl + 2	Reset the camera to view from the left perspective		
Ctrl + 3	Reset the camera to view from the right perspective		
Ctrl + 4	Reset the camera to view from the back perspective		
Ctrl + 5	Reset the camera to view from the top perspective		
Ctrl + 6	Reset the camera to view from the bottom perspective		
Ctrl + 7	Reset the camera to view from the isometric perspective		
Ctrl + S	Toggle the structure view type between solid and contour slice		
Aperture Editing			
Key	Description		
Ctrl + Z	Undo the last edit (only tracks edits to undo while in the same instance of the task)		
ESC	Stop drawing manual edits (during line drawing)		

General	
Optimization Navigation	
Key	Description
Left/Right Arrow	Move mco slider in 1/10 increment (applies to last changed slider)

From:

http://apps.dotdecimal.com/ - decimal App Documentation

Permanent link:

http://apps.dotdecimal.com/doku.php?id=planning:userguide:userguide:keyboard\_controls&rev=1486046534

Last update: 2021/07/29 18:25

